

## Donning and Duffing a respirator mask

Description:

**PLEASE ENSURE THAT THE MEDICAL EVALUATION HAS BEEN COMPLETED BEFORE DONNING MASK, THIS CAN BE COMPLETED BY A REGISTERED NURSE.**

Met	Not Met	Title	Description	Rationale
<input type="checkbox"/>	<input type="checkbox"/>	Opening package	Inspect respirator including straps for tears or damage. If damage replace it.	
<input type="checkbox"/>	<input type="checkbox"/>	Opening mask	If folded flat open folds fully following manufactures recommendations.	
<input type="checkbox"/>	<input type="checkbox"/>	Donning mask	Place respirator on your face with nose piece at fingertips, allowing the headbands to hang freely.	
<input type="checkbox"/>	<input type="checkbox"/>	Donning Mask	The respirator should cup your chin. Pull the top strap over your head resting high at back of head, The bottom strap is placed around your neck and below your ears. DO NOT CRISS CROSS THE STRAPS.	
<input type="checkbox"/>	<input type="checkbox"/>	Donning mask	Make sure your mouth and nose are covered by respirator. If a metal nosepiece is present, use both hands to mold the nose strip to conform to the shape of your nose, by pushing inward while moving fingertips down along the nosepiece.	
<input type="checkbox"/>	<input type="checkbox"/>	Seal check	Cover surface of the respirator with hands so air is prevented from passing through the filtering facepiece.	
<input type="checkbox"/>	<input type="checkbox"/>	Positive pressure seal check	Gentle exhale to see if facepiece bulges slightly	
<input type="checkbox"/>	<input type="checkbox"/>	Negative pressure seal check	Take a quick deep breath to see if facepiece collapses slightly. If air escapes during either test the facepiece does not fit properly. Do not wear mask if seal can not be obtained.	
<input type="checkbox"/>	<input type="checkbox"/>	Duffing mask	Remove facepiece carefully and place in bag or Tupperware container with your name on it for next use. Each employee should have there own mask that they can use up to 5 of their shifts.	

Donning and Duffing a respirator mask Requirements Met

Donning and Duffing a respirator mask Requirements NOT Met

**Observer's Notes**

\_\_\_\_\_  
Learner's Name

\_\_\_\_\_  
Learner's Signature

\_\_\_\_\_  
Observer's Name

\_\_\_\_\_  
Observer's Signature

\_\_\_\_\_  
Observation Date

---

**Suggested Courses**

Course Code	Title	Course Type
-------------	-------	-------------

---

**Approved Observers**

Name [User Name]	Hierarchy Name	Location	Email
---------------------	----------------	----------	-------